v1.4.16

Please keep the instructions for warranty purpose

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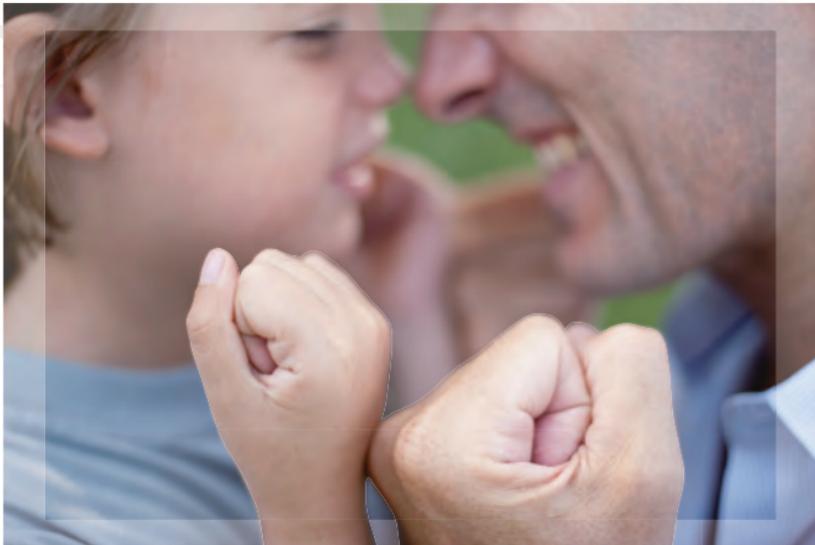
REGAL Prosthesis Ltd. 皇室義肢有限公司

An ISO 13485:2016 and ISO 9001:2015 certified company
Your reliable partner

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Silicone Cosmetic Prostheses instructions English

1. Applications

- REGAL PROSTHESIS LTD Silicone Cosmetic Prostheses are used in providing aesthetic appearance to amputees so as to build up their confidence and to bring joy and happiness to their loved ones.

2. Indications

- Model 101L, 102L, 102LS – For above wrist amputation
- Model 101LL, 102LL, 102LSL – For above wrist amputation (long sleeve).
- Model 103 – For partial hand amputation
- Model 100 – For both myo-electric and mechanical hands.
- Model 100L - For both myo-electric and mechanical hands (long sleeve).
- Model 100PL – For pediatric myo-electric hand.
- Model 200 series – For finger amputation.
- Model HDSFHT, HDSFHT-ER, HDSF, HDSF-ER – For partial foot amputation.
- Model TOE series – For Toe amputation
- Model BKSL – For covering leg amputation and for covering lower limbs prosthesis respectively.
- Model SL – Silicone arm sleeve for covering forearm
- Semi-Custom Made (SCM) – Adaption to residual limb, applies to all above Models.

3. Contraindications

- The following Models should not be used in the presence of open wound: Model 103, 200 series, HDSFHT, HDSFHT-ER, HDSF, HDSF-ER, TOE series, BKSL and SL;

4. Precautions

- Our Silicone Cosmetic Prostheses should be fitted by a certified prosthetist who is familiar with the purpose for which

they are prescribed. The prosthetist is responsible for providing wearing instructions and precautions to the amputees.

- Contact your prosthetist if:

- Unusual swelling, skin irritation occurs, stop using the product and consult with the prosthetist immediately;
- The Silicone Cosmetic Prosthesis has split or any damage has occurred;
- Any of the inner structure such as finger wires or wrist connector is broken which may cause injury or damage the prosthesis with continued use;
- The prosthetic socket is broken which may cause injury or damage the Silicone Cosmetic Prosthesis with continued use

5. Instructions for Use

- Model 101L, 102L, 102LS, 101LL, 102LL, 102LSL:

Apply water-based lotion onto the prosthetic socket and gently slide the glove onto it. Consult with your prosthetist for arm socket). For Model 102L and 102LL, epoxy glue can be used at the bottom end of the socket to attach onto the pre-filled foam.

- Model 100, 100L and 100PL:

Apply water-based lotion inside the glove surface before installing the glove on the myo-electric or mechanical hand.

- Model 103, 200 series, HDSFHT, HDSFHT-ER, HDSF,

HDSF-ER, TOE series, BKSL and SL:

Apply water-based lotion onto the residual limb before donning the prosthesis.

6. Care and Cleaning

- Keep the silicone prosthesis away from fire, solvents and oil based liquid;
- Avoid using sharp objects such as knives or scissors as these

could cut and tear the prosthesis easily;

- Do NOT expose the prosthesis to sunlight for extended periods, this could cause the color to fade;
- DO NOT use excessive force to pull residual fingers from the silicone prosthesis, this may tear the prosthesis. The prosthesis may create a vacuum between the residual limb and the prosthesis inner surface, in this case, simply pinch the prosthesis and make an air passage to eliminate vacuum before removing the prosthesis. Alternatively the patient may apply water-based lotion on the outside surface of the finger prosthesis as lubrication then roll the prosthesis inside out to remove;
- Remove stains immediately with mild soap and water. Stains may become permanent if not cleaned immediately, our warranty does not cover such incidents;
- For hygiene purposes, we recommend to cleaning your prosthesis daily with mild soap and water;
- Make use of small items such as bandages, rings, bracelets, sports wrist band or watches to camouflage the prosthesis edge;
- When using a silicone prosthesis with zipper, please ensure the zipper is FULLY opened before donning and doffing. If the zipper is not fully opened while donning or doffing the zipper can be damaged easily;
- For Model 102LS and 102LSL (silicone prosthesis with integrated finger wires and wrist connector): DO NOT apply force such as using the prosthesis to support or to lift the body. Doing so will damage the finger wires and injury may result from broken finger wires caused by misuse. Our silicone prostheses are designed for cosmetic purposes only;
- For Model 102LS and 102LSL: DO NOT take the inner foam hand out of the prosthesis as this will void the warranty.

7. Warranty and Return Policy

- Warranty may be void if any of the "Care and Cleaning" recommendations above are not followed. For more details about our warranty and return policy, please refer to our catalogue 2015.

8. Declaration of Conformity

- Regal Prosthesis Limited, as the manufacturer of silicone prostheses, declares that our silicone prostheses conform to the requirements of European Community Directive 93/42/ EEC.
- Keep this manual in a safe place for future reference.

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